

Operation Manual

Components Layout

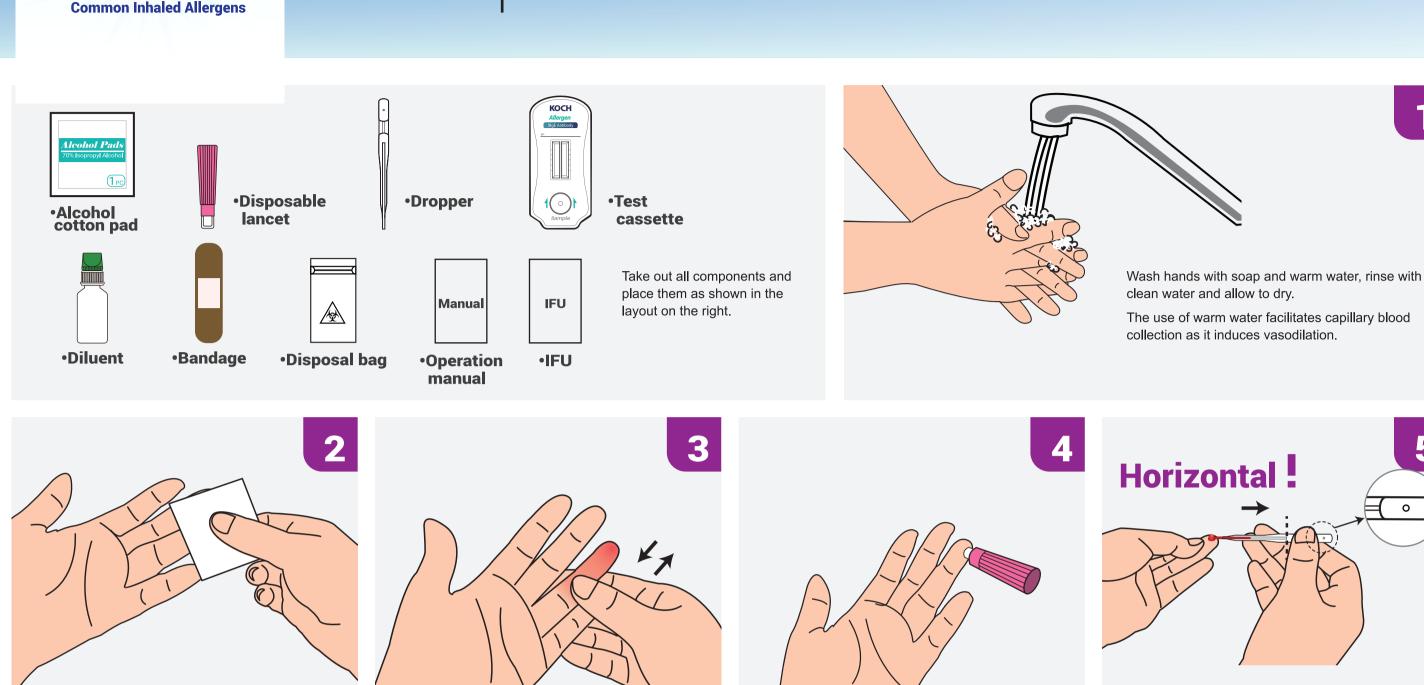

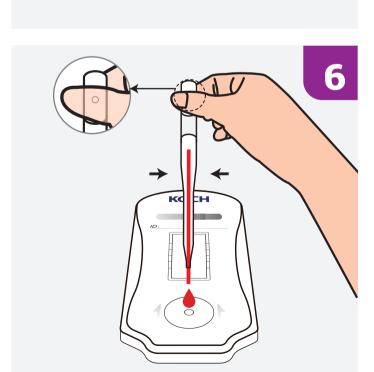


Use alcohol cotton pad to wipe your 3rd(middle) or

4th(ring) finger.

Once the dropper is filled, please block the hole in the bulb with your fingers unless the blood would be leaked, then squeeze the blood collected into the sample hole on the cassette within 30 seconds.

During this process, please keep the hole in the bulb blocked.

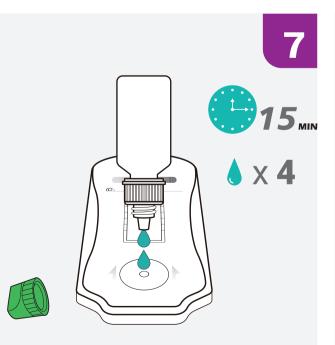

Gentlely massage of the hand / wrist / finger or

getting the blood to flow and move down to the

fingertip is important for proper blood collection.

immersion them in warm water to warm your finger,

Unscrew the green cap and squeeze 4 drops of buffer into the sample hole. Wait 15min and read results.



Children or teenagers below 18 years old should be pricked by adults.

Apply a bandage on the wound after blood

collection.



Hold the dropper horizontally without pressing the bulb, keep the hole in the bulb exposed.

Rub the finger, contact blood drop with dropper tip horizontally, wait blood enter the inner dropper, this process would repeat for several times until the inner dropper is filled.

During this process, please keep the dropper horizongtally. Once the dropper is filled, please block the hole in the bulb with your finger unless the blood would be leaked.

e1 w6 t3 g6 i6

